Transferhilfe Merlin Transfer Aid Merlin

Gebrauchsanleitung · Manual
Bruksanvisning · Käsikirja

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer aids.

The use of Transfer Aid Merlin should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Transfer Aid Merlin is part of a product line which is used specifically to assist patients with restricted mobility and a height of 140 cm to 200 cm to stand up and transfer in hospitals and other health care facilities, as well as in home care.

The product may only be used for the specified purpose and under the specified conditions.

The Transfer Aid Merlin facilitates easier, energy-saving and back-friendly transfers. Remaining strength in the patients arms and legs can be mobilised. The patient experiences a strong sense of security during the transfer. The patient should be able to hold on and to assist with standing up.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: **PMM-1100** Transfer Aid Merlin

1.2 Accessories

Item no.: PM-6160 Raising Aid Merlin

Item no.: PM-3010 Loop Belt

Item no.: PMM-1150 Casters Set

Item no.: PMM-1120 Leg Pad

1.3 Scope of delivery

Transfer Aid Merlin PMM-1100

User manual

1.4 Storage, transport

Store/transport at -5 C – 40 C

Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Before using the Transfer Aid Merlin to help the patient stand up, ensure that the patient is not suffering from any injuries or weakness which would preclude use of the device.
- For safety reasons, the movement must always be secured by an assistant
- The Transfer Aid Merlin PMM-1100 may only be used by trained persons, or respectively by persons who have been instructed in use of the device and who are familiar with the patient's medical condition.
- The user / assistant must have sufficient strength to perform the transfer and to support the patient if necessary
- The device may only be used on solid, even surfaces
- It is essential to ensure that the patient's centre of gravity remains within the bottom plate area of the Transfer Aid Merlin.
- In case of proximity to stairs and exits, ensuring sufficient distance and securing the transfer is essential
- The screw connections and tilting lever of the Transfer Aid Merlin must be inspected at regular intervals to ensure that they are tight
- The use of connectors from third-party providers is prohibited. Please use only original connectors (screws, etc.)
- Use of the Transfer Aid Merlin must be prevented in the event of noticeable damage to the frame or rollers
- In the event of wear or damage to the device which cannot be repaired, the Transfer Aid Merlin must be taken out of service
- If the transfer with the Transfer Aid Merlin PMM-1100 isn't secured with the belt, it is necessary to ensure that the patient is able to stand securely. Keep the transfer distance as short as possible.

- Both brakes must be engaged during positioning and transferring, in order to prevent the Transfer Aid Merlin from sliding away
- When the Transfer Aid Merlin is not in use, ensure that it remains in an area where neither patients nor caregivers can trip over the bottom plate
- Don't use and sharp-edged or abrasive objects and don't use any solvents for cleaning
- Please observe the storage instructions under item 1, in order to preclude potential external sources of heat or cold which could lead to a change in the surface temperature
- The Transfer Aid Merlin PMM-1100 must not be stored in a location where it is exposed to sunlight for long periods. Extended exposure to sunlight can heat up the stainless steel frame and result in skin injuries. Likewise, the Transfer Aid Merlin PMM-1100 must not be stored in a cold storage room or similar spaces with sub-zero temperatures, since this could also result in skin injuries when coming in contact with the stainless steel frame
- The Transfer Aid Merlin PMM-1100 may be used exclusively as a raising aid in accordance with the intended use. Any use other than the intended use is prohibited
- Support Belt Merlin (item no. PM-6160): Use of the Support Belt is limited in patients with contracture of the legs
- Not suitable for people who are less than 146 cm tall, weigh less than 40 kg or have a BMI of less than 17.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

2.2 Function check

The Petermann Transfer Aid Merlin PMM-1100 is designed with a stainless steel frame and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. The device may not be used if there are any noticeable defects.

The screw connections and tilting levers must be inspected at regular intervals to ensure that they are tight.

Periodic inspections should be performed at least once per year; i.e. visual inspection of the load-bearing parts (function test). For sanitary reasons, the device must be cleaned regularly.

- **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Transfer Aid Merlin is suitable for all applications involving people who require assistance to stand up and transfer.

The Transfer Aid Merlin PMM-1100 can be used:

- To ensure a high level of patient safety when transferring a patient, even with elderly, weaker or insecure people.
- Grip the handles for safety, is possible in several places
- To achieve additional stability during the transfer
- To preserve mobility, for example when using the toilet
- For Dressing and undressing the upper body in a standing position
- Toilet hygiene in a standing safe position
- The use of motorized lifts is less necessary, you are independent of power sources
- Support for all people who cannot allow physical closeness
- Support for all people who have cognitive disorders of various origins
- Support for severely multiple disabled persons
- Less personal support is required
- Note: A professionally trained person must decide whether the Transfer Aid Merlin may be used and possibly add the accessory Support Belt Merlin

The Transfer Aid Merlin is designed to be reusable! How often the Transfer Aid Merlin can be reused depends on the condition of the respective device. On the basis of the MPG (German Medical Devices Act) "Processing of Medical Devices" § 4, para. 2 MPBetreibV (Ordinance on Operators of Medical Devices). If applicable, please observe the corresponding information for the Support Belt and/or belts in the separate user manuals.

The device can be cleaned and/or disinfected as follows before being used again:

- Proper preparation, pre-cleaning, disassembly of the device
- Visual inspection for cleanliness and integrity (e.g. material properties, corrosion)
- Cleaning, rinsing and drying
- Function test of the base frame and the rollers, adjustability of the leg pad
- General repairs: Replacing accessories as required
- Final function testing of the device
- In the event of wear or damage to the device which cannot be repaired, the Transfer Aid Merlin must be taken out of service.

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

The Transfer Aid Merlin PMM-1100 cannot be used with the following clinical pictures:

- Patients with a complete lack of muscle tone
- Patients with extremely increased muscle tension or contractures that preclude joint movements
- Sedation
- Height less than 146 cm
- Weight less than 40 kg or BMI less than 17
- Height greater than 200 cm
- Cognitive disorders which are associated with defensive physical reactions

3.3 Application risks

- Untrained user ► risk of tipping/falling
- Incorrect assessment of the patient's ability ► risk of falling
- Lack of instruction and risk assessment ► risk of tipping/falling

- Patient lacks physical stability ► risk of tipping
- Driving around sharp turns on over inclined surfaces ► risk of tipping
- Uneven and unpaved floor coverings ► risk of tipping
- Brakes not locked ► risk of falling
- Transfer without Support Belt Merlin ► risk of falling (it is necessary to ensure that the patient can safely come to a stop on their own. Keep the transfer distance as short as possible!)

Please observe the safety information under item 2


3.4 Areas of application

The product can be used in hospitals and other health care facilities as well as in home care settings.

The Petermann Transfer Aid Merlin assists with:

- Transfers (e.g. bed → wheelchair – wheelchair → toilet)
- Mobilisation to standing
- Preparing to go to the toilet
- removing clothes while standing
- performing personal hygiene while standing


3.5 Application illustration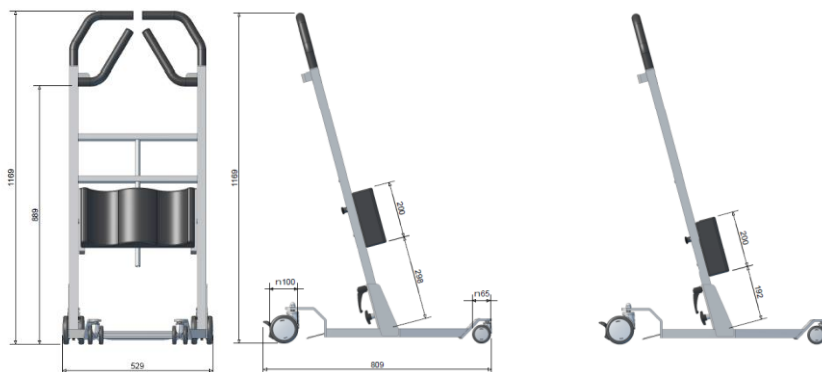

3.6 Application

3.6.1 General description


The Transfer Aid Merlin consists of a stable platform (3) to support the feet, which is bordered by a frame. The frame tapers towards the rear. The front of the platform is inclined down. Two swivel casters without brakes are mounted on the front of the frame. Two swivel casters with brakes are located on the back of the frame. The casters are suitable for use in wet rooms. The casters are not suitable for use on uneven surfaces under load. The platform is coated with an anti-slip material. A stainless steel frame (2) is attached to the platform and can be removed from the platform during transport. Height-adjustable leg supports are located near the bottom (1). The rest of the frame is designed to provide many different handhold options. The upper part of the frame has skin-friendly padding on the handle ends. A hook to attach a Support Belt is located on the frame.

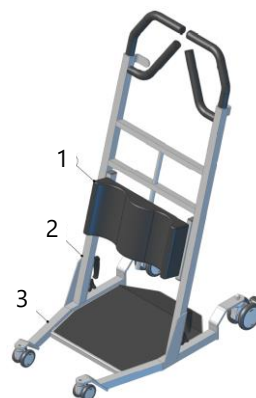3.6.2 Assembly

The Transfer Aid Merlin PMM-1100 can be taken apart and is delivered disassembled.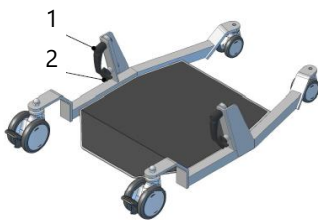

- Insert the upper frame into the receiver on the bottom platform.
- Secure it by tightening the screw connections and tilting levers.
- Please note! Tighten the handle screws (2) **first!**
- Then fix the tilting lever (1)
- The screw connections and tilting lever must be inspected at regular intervals to ensure that they are tight.

The Transfer Aid Merlin can be broken down into two pieces for transport. To remove the upper frame from the bottom platform, the two tilting levers (1) and the handle screws (2) must be released.

Engaging both brakes is essential when transporting the platform, in order to prevent the platform from rolling away.

3.6.3 Preparing for the transfer

- Position the patient's feet on the base plate.
- Adjust the leg pad to the correct height –upper edge approx. 3 finger widths below the patella.
- **The lower legs must be within the supports, otherwise safe use is impossible!**
- Determine the grasping height on the device – approx. shoulder height of the upright sitting patient. 
- The handle positioning allows for four different grasping heights.
- The lower the grasping height, the less force the patient can apply with their arms.
- Ensure that the patient positions their hands correctly! The palms must face down while grasping, otherwise opposing movements result.
- The patient pulls the device towards them.
- This brings the patient's lower legs into an optimal position for standing up.
- **The two brakes are only engaged once this position has been reached!** 

3.6.4 Application without Support Belt and/or belt

The Transfer Aid Merlin PMM-1100 without Support Belt is used primarily for people who only require a bit of assistance to stand up from a sitting position.

Preparation see 3.6.3

- The patient stands up, with assistance if necessary.
- Release the brakes.
- Now the Transfer Aid Merlin can easily be pulled or pushed to a different location.
- **If the patient is not secured by a belt, then the distances should be kept short.** 
- Position the Transfer Aid Merlin close to and centred in front of the new sitting location. 
- **Engage the brakes!**
- The assistant positions themselves beside the patient.
- Check whether the patient is grasping tightly.
- If the "target seat" is significantly lower than the "starting seat", position the patient's hands one handle bar lower.
- The patient sits down slowly.
- Support the patient's lower back if necessary.
- Once the patient is sitting safely, release the brakes.
- Roll the Transfer Aid Merlin back about one foot.
- Engage the brakes.
- Move the patient's feet from the platform onto the ground.

3.6.5 Application with Support Belt

The Transfer Aid Merlin PMM-1100 with the Alpha® Support Belt Merlin (Item-No.: PM-6160) is used primarily for people who can no longer get up on their own but still have sufficient muscle tone.

Use of the Support Belt is limited in patients with contracture of the legs.

Preparation see 3.6.3

- Place the Support Belt Merlin around the sitting patient's waist.
- Close the inner part of the belt and pull it tight.
- Lay the outer belt onto the patient's legs.
- Fasten the looped belt (PM-3010) if necessary. (see point 3.7)
- Prepare for the transfer (see point 3.6.3)

The assistant stands before the Transfer Aid Merlin.

- The assistant grasps the two outer belts of the Support Belt Merlin on the patients legs.
- The assistant places one foot on the inclined surface of the Support Belt Merlin base plate.
- The assistant shifts their weight back and pulls the Support Belt towards them.
- The patient's movement is supported as they stand up.
- Hook the inner loops of the Support Belt Merlin onto the side hooks of the Transfer Aid Merlin in the required position.
- Select the shortest possible belt adjustment for the transfer.
- Secure the standing patient with their pelvis as close as possible to the device.
- Release the brakes.
- Now the Transfer Aid Merlin can easily be pulled or pushed to a different location.

The assistant stands to one side

- Grasp the rear handle of the Support Belt Merlin with one hand.
- Find the optimal pressure point for one of the three handles on the rear belt. (Support Belt Merlin)
- Use the other hand to generate opposing pressure from the upper handle on the device.
- The patient's movement is supported as they stand up.
- Hook the inner loops of the Support Belt Merlin onto the side hooks of the Transfer Aid Merlin in the required position.
- Select the shortest possible belt adjustment for the transfer.
- Secure the standing patient with their pelvis as close as possible to the device.
- Release the brakes.
- Now the Transfer Aid Merlin can easily be pulled or pushed to a different location.

3.6.6 Assistance during sitting

- Push the Transfer Aid Merlin directly in front of the destination
- Keep the distance between the patient's calves and the destination low
- Engage the brakes on the Transfer Aid Merlin
- Bring the patient to the highest standing position by providing supporting pressure in the lower back area
- There is no tension on the loops of the Support Belt Merlin
- If the belt slipping up is a concern, then the looped belt (PM-3010) must be tightened.
- The assistant stands beside the patient
- The Patient holds on to the handle of the Transfer Aid Merlin.
- Unhook the loops of the Support Belt Merlin from the hooks, starting with the outer loop
- Support the patient in the standing position and during sitting, from the side or from the front, as required.
- The belt is removed once the patient is sitting safely
- Roll the Transfer Aid Merlin back about one foot
- Engage the brakes
- Move the patient's feet from the platform onto the ground

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.6.7 Behaviour in case of a fault

The following information helps you with troubleshooting:

Fault	Possible cause	Fault resolution
Merlin is hard to move	Casters defective / dirty / gummed up	Clean, if necessary replace the casters

If you are unable to rectify the fault despite this guide, please contact your responsible medical supply store or call our service number: **+49 (0) 9868 9339-0**

3.7 Approved accessories, combination with other products, sets

Support Belt Merlin PM-6160

The Support Belt Merlin is a multi-function belt and can be used as an optional accessory with the Transfer Aid Merlin PMM-1100.

The Support Belt Merlin consists of a padded inner belt with front-facing buckle. The Support Belt Merlin can be continuously adjusted to a circumference of 70 cm - 160 cm. The belt widens from 10 cm to 20 cm at the back. Three horizontal handles are located on the belt of the Support Belt Merlin.

The outer belt is equipped with four double handle loops per side. These can be used both as pulling loops and as attachment loops.

Please observe the separate user manual for the Support Belt Merlin (item no.: PM-6160).

Looped belt item no.: PM-3010

The looped belt can optionally be attached to the loop closest to the body on the outer belt to act as an additional belt.

Petermann Transfer Aids Merlin are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

Artikel-Nr. Set	Bestandteile Set
PMM-1000	PMM-1100 Transfer Aid Merlin
	PM-6160 Support Belt Merlin
	PM-3010 Looped Belt

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Transfer Aid Merlin is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

The Transfer Aid Merlin rating plate is located on the back of the base plate. For inquiries, please provide the LOT number on the rating plate.

5 Cleaning, care and disinfection

5.1 Base frame

The metal frame of the Transfer Aid Merlin is wiped with a normal dust cloth. You can use common liquid household cleaner. The Transfer Aid Merlin can also be treated with an off-the-shelf wipe disinfectant. Verify compatibility first, if applicable according to the disinfectant information. We recommend aldehyde and alcohol free products such as Trionic®, Mikrobac® Tissues, Mikrobac® forte, Bacillol® 30 Foam or Bacillol® 30 Tissues.

5.2 Leg pad

Soft polyurethane foam can be disinfected with a max. 60 % ethanol or propanol solution. Other disinfectants with acids or corrosive/damaging tensides are not recommended.

Recommended products:

Micronova
Terralin *liquid*
antifect N *liquid*
Meliseptol
ClearSurf

Please note that these surface disinfectants are applied using wipe disinfection.

5.3 Other

Clean the casters on a regular basis and ensure that they are turning freely.

Observe the instructions in the separate user manuals of the Support Belt and/or belts.

If you have further questions or don't wish to perform the inspection, cleaning and disinfection of your Transfer Aid Merlin yourself, please call our service number.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PMM-1100.2	Stainless steel	grey	200 kg
PMM-1120	PU-foam	black	200 kg
Base plate	Thick film, roughened	black	200 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PMM-1100.2	Transfer Aid Merlin	529 x 809 x 1169 mm See picture 3.5	21,8 kg	(1)04250355311009(10)x
PMM-1120	Leg pad	400 x 200 x 75 mm adjustable from 192 – 498 mm		
	Base-plate	410 x 465 x 57/97 mm		
	Grip area height	Von 889 – 1169 mm		
	Castors in front	Ø 65 mm		
	Castors on behind	Ø 100 mm		
	Turning cycle	≥ 850 mm		

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning!**

Please note that the warranty does **not** apply to:

- Repairs which were not performed by Petermann or authorised Petermann dealers.
- Accessories such as casters and belts.
- Damage to or failure of the device caused by improper use.
- Accidental or indirect injury and/or damage to property.

8 Disposal information

There are no specific disposal regulations for Transfer Aids Merlin, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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