Maxiglide

Maxiglide Sliding Board

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Bruksanvisning · Käsikirja

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer aids.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Maxiglide is part of a product line which is used specifically to support the repositioning and/or transfer of patients in hospitals and other health care facilities. as well as in home care.

The Maxiglide is a safe transfer aid, which facilitates transfers from bed to bed or other recumbent positions in a manner pleasant and comfortable for the patient as well as simple and back-friendly for the users. The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

The product may only be used for the specified purpose and under the specified conditions.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: **BM-5060 Maxiglide**

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Maxiglide
User manual

1.4 Storage, transport

Store/transport at -5 C – 40 C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **five years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Weight limits must be observed. (170/200 kg see point 6)
- The handles are intended solely for positioning. Do **not** reach into the handle openings during transfer.
- The use of non-standard cleaning agents or bleaching agents, alcohols or other chemical additives may lead to permanent deterioration of the sliding mat properties.
- Ensure that both ends of the sliding board are securely supported on the transfer surfaces. The clear distance bridged with the sliding board must not exceed 10/20 cm. (see point 6)
- Transfers under naked skin must be aided with a protective layer (e.g. towel).
- The smooth surface significantly reduces resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label
	The distance between free transfer surfaces using the sliding board must not be more than 10/20 cm (see point 6)		

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

Stability test

Bend product to test stability

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since the Maxiglide is suitable for all applications involving people who require assistance during sideways transfers.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

3.3 Application risks

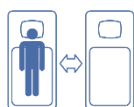
Untrained user

The Maxiglide may be lifted and/or pushed incorrectly if the user hasn't been trained in or is unfamiliar with correct use of the Maxiglide. In addition, movements may be performed too quickly or jerkily. This could damage the health of the patient and/or user. Observe the safety information under item 2.

- Insufficient care while placing under without draw sheet → risk of pinching the skin
- Lack of stabilisation → risk of falling

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

3.5 Application illustration

Not available

3.6 Application

3.6.1 Placement

- Place the patient on their side in a position their medical condition allows. (ideally with the draw sheet).
- Place Maxiglide between bed sheet and draw sheet (hips and shoulders should lie on Maxiglide)
- Place the Maxiglide between the bed sheet and the draw sheet under the patient
- Place the patient on their back.

3.6.2 Transfer methods

- Important: Before the transfer, make sure that the Maxiglide rests evenly on both lying surfaces
- Work beside the bed.
- A draw sheet supports the transfer and enables a back-friendly working method
- Pull the patient in the desired transfer direction with the draw sheet
- Without a draw sheet: slide the patient over the Maxiglide

3.6.3 Removal

- Briefly place the patient slightly on their side.
- Remove the Maxiglide from under the patient.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

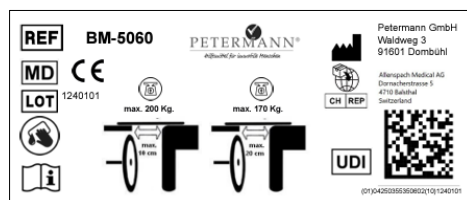
3.7 Approved accessories, combination with other products

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

4 Product marking / CE

The Petermann Maxiglide is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

Cleaning	Disinfection (if necessary)
Damp cleaning	Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues*
Warm water / mild detergent	
Do not use abrasive or bleaching agents!	
Do not use sharp-edged objects!	* wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection.

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
BM-5060	Polyethylen, latex free	black	200 kg with 10 cm free transfer surface
			170 kg with 20 cm free transfer surface

Item no.	Description	Dimensions in mm	Weight	UDI
BM-5060	Maxiglide	1500 x 600 x 6	4000 g	(01)04250355350602(10)x

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Maxiglides, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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